

Cover Page



Janani Family Care Pvt. Ltd.

A SUBSIDIARY OF
dkt
INTERNATIONAL

PRODUCT CATALOGUE

Let Every Child Be Wanted



This catalogue is intended for pharmacists and healthcare providers only.

Trust
CONTRACEPTION

STYLE
CONDOMS

**COMBI
CLEAR**

safe-t kit
Abortion Kit

MVA[®]

About Us

Janani Family Care Pvt Ltd, a subsidiary of DKT International, is a pioneer of family planning and reproductive products in India. Our mission is to increase the availability and affordability of a full basket of modern family planning and reproductive health methods by strengthening the supply chain and increase demand through media and educational campaigns. We have a vision of being an innovative and adventurous pharmaceutical enterprise that improves people's lives.



Innov8 CP2, 44, Backary Portion,
2nd Floor, Regal Building, Connaught Place, New Delhi, 110001

STYLE

CONDOMS

STYLE RANGE

STYLE
PREMIUM CONDOMS

☐ Premium Condoms

STYLE
ULTRA CONDOMS

☐ Ultra Condoms

STYLE
EXCLUSIVE CONDOMS

☐ Exclusive Condoms

STYLE

PREMIUM CONDOMS



STYLE PREMIUM (CHOCOLATE)

Feature: Dotted and chocolate scent.
M.R.P.: 3s - 30/-, 10s - 90/-

STYLE PREMIUM (MANGO)

Feature: Dotted and mango scent.
M.R.P.: 3s - 30/-, 10s - 90/-



Pack Size 3s - 48 packs per box
10s - 12 packs per box

STYLE

PREMIUM CONDOMS



STYLE PREMIUM (STRAWBERRY)

Feature: Dotted and strawberry scent.
M.R.P.: 3s - 30/-, 10s - 90/-

STYLE PREMIUM (VANILLA)

Feature: Dotted and vanilla scent.
M.R.P.: 3s - 30/-, 10s - 90/-



Pack Size 3s - 48 packs per box
10s - 12 packs per box

STYLE

PREMIUM CONDOMS



STYLE PREMIUM (COSMO)

Feature: Dotted and mixed fruit scent.
M.R.P.: 3s - 30/-, 10s - 90/-

Pack Size 3s - 48 packs per box
10s - 12 packs per box

STYLE

ULTRA CONDOMS



STYLE ULTRA (COFFEE)

Feature: Dotted and coffee scent.
M.R.P.: 3s - 35/-, 10s - 110/-

STYLE ULTRA (JASMINE)

Feature: Dotted and jasmine scent.
M.R.P.: 3s - 35/-, 10s - 110/-



Pack Size 3s - 48 packs per box
10s - 12 packs per box

STYLE

ULTRA CONDOMS



STYLE ULTRA (LAVENDER)

Feature: Dotted and lavender scent.

M.R.P. : 3s - 35/-, 10s - 110/-

Pack Size 3s - 48 packs per box
10s - 12 packs per box

STYLE

EXCLUSIVE CONDOMS



STYLE EXCLUSIVE (AIR THIN)

Feature: Condom with reduced thickness for greater sensitivity

M.R.P. : 3s - 40/-, 10s - 120/-

STYLE EXCLUSIVE (EXTRA TIME)

Feature: Dotted with delay effect to last longer

M.R.P. : 3s - 40/-, 10s - 120/-



Pack Size 3s - 48 packs per box
10s - 12 packs per box

STYLE **Vig**ueur

STYLE Vigueur is a high-quality brand of sildenafil that induces hard erections which can last for an extended duration thereby enhancing confidence, well-being, and satisfaction. **STYLE Vigueur** can help to manage erectile dysfunction (ED), a condition experienced by many men which means it is difficult to achieve or sustain an erection hard enough for sex.



**Sildenafil Tablet IP
100 mg**

Pack Size **10 packs per box**

Confidence in Every Dose

*conditions applied, speak with a doctor

HOW IT WORKS

STYLE Vigueur increases the amount of blood that flows to the penis. This causes hard erections which can last for an extended duration. It works with sexual stimulation to achieve a completely hard and fully rigid penis (GRADE 4 ERECTION).

HOW TO USE

- **STYLE Vigueur** should be used after consultation with a healthcare provider
- 50 mg to 100 mg is the recommended dosage that should be taken only once in a given 24 hour period
- The dose is taken 30 - 40 minutes before sexual activity
- Avoid consumption of alcohol and fatty foods at the same time as taking **STYLE Vigueur** to optimize drug absorption
- Never exceed the prescribed / recommended dose

WHO CAN USE

- Men who have difficulty getting or sustaining an erection (erectile dysfunction) and want to enjoy a satisfying sexual experience

WHO CANNOT USE

- Men below the age of 18 years and females
- Men with medical conditions e.g liver dysfunction, kidney dysfunction, hypotension, recent stroke or heart attack


- Men taking certain medications e.g nitroglycerin, sodium nitroprusside, amyl nitrite, erythromycin and cimetidine

BENEFITS

- Enhances confidence in men who are unable to achieve and/or maintain an erection
- Delivers satisfaction for both partners in sexual activity
- Good user satisfaction for up to 6 hours
- Taken only when needed, no regular administration required to maintain effect


COMMON SIDE EFFECTS

STYLE Vigueur side effects are often mild and go away on their own. They include, nausea, headache, diarrhea, chest pain, muscle ache, allergic reaction, prolonged erection and nasal congestion.

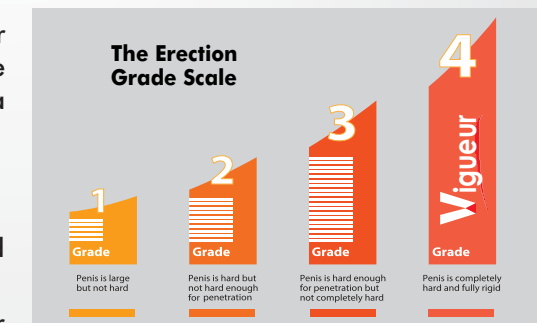

Trust

Your Family Planning Decisions

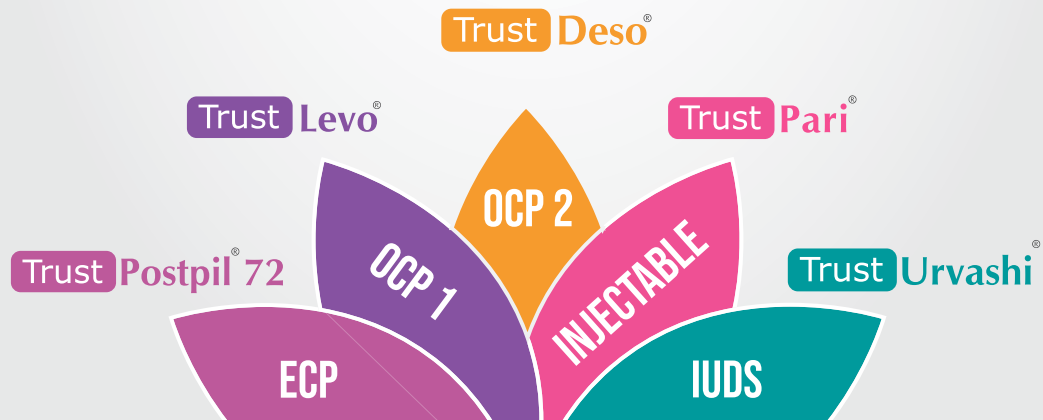

Rapid Pregnancy Detection Kit

Trust Sure[™]
Your family planning decisions



Trust Sure is a rapid test for detection of hCG hormone in female urine. hCG is a hormone that rapidly increases in the body in the first few weeks of pregnancy. **Trust Sure** is sensitive to 25mIU/ml and so confirms a positive pregnancy following a missed menstrual cycle.



Pack Size 50 cassettes per box

Levonorgestrel Tablets IP

Trust Postpil 72[®]
Emergency Contraceptive Pill



Trust Postpil 72 is an effective emergency contraceptive pill that can prevent pregnancy after unprotected sex. **Trust Postpil 72** is one tablet of 1.5 mg of Levonorgestrel and should be taken within 72 hours of unprotected sex.



Pack Size 12 packs per box

One Pill Within 72 Hrs And Be Stress Free

*conditions applied, speak with a doctor

HOW IT WORKS

Trust Postpil 72 works by temporarily delaying ovulation. It also slows down the active forward movement of sperm by thickening the cervical mucus.

HOW TO USE

Trust Postpil 72 is more effective the sooner it is taken following unprotected sex, but within 72 hours. **Trust Postpil 72** is taken orally as a single dose tablet.

WHO CAN USE

Most women following unprotected sex who want to prevent pregnancy

WHO CANNOT USE

Trust Postpil 72 is not suitable for pregnant women.

WHEN TO USE

Trust Postpil 72 can be used by a woman:

- Following unprotected sex
- Not on any regular method of contraception
- Who relies on traditional methods of contraception such as calendar based or withdrawal methods
- Following Inconsistent use of contraceptives, for example forgot to take regular contraception pill within required time frame

BENEFITS

- **Trust Postpil 72** is safe for use in breastfeeding mothers whose baby is 6 weeks of age or older
- Available in pharmacies and chemist stores
- Quick and easy option for preventing unplanned pregnancy
- Can offer peace of mind

COMMON SIDE EFFECTS

Not everyone will experience side effects. They are often minimal and go away quickly and may include

- Nausea
- Headache
- Dizziness
- Breast tenderness
- Lower abdominal discomfort
- Menstrual change (period may come early or late)

Rx
Levonorgestrel & Ethinylestradiol Tablets IP
with Ferrous Fumarate Tablets IP

Trust Levo[®]

Birth Control Pills
with Iron



Trust Levo is a safe and effective oral contraceptive pill that is used to prevent pregnancy when taken every day. Each pack contains 21 active white tablets (Levonorgestrel 0.15 mg + Ethinylestradiol 0.03 mg) and 7 brown tablets (Ferrous Fumarate 60 mg). Besides pregnancy prevention, **Trust Levo** can help regulate the menstrual cycle.



Pack Size 1 Cycle - 15 packs per box
2 Cycle - 12 packs per box

Birth Control Pills Protection All Day Every Day

*conditions applied, speak with a doctor

HOW IT WORKS

Trust Levo works in two ways:

- By preventing maturation of the egg
- By thickening the cervical mucus, so sperm cannot reach the egg

HOW TO USE

Trust Levo is very easy to use: 1 pill should be taken at the same time every day or within the same 12 hour window every day for 28 days then a new pack is started immediately. **Trust Levo** will be more effective if taken at the same time every day.

WHO CAN USE

Trust Levo is suitable for women:

- Women who want a short-term contraceptive method
- Breast-feeding mothers (from 6 months after childbirth)
- Women with irregular menstrual cycle
- Women with anaemia as a result of heavy menstrual bleeding
- Women who suffer with lethargy during their period

WHO CANNOT USE

Trust Levo is not suitable for women with certain health conditions, including:

- Sickle-cell anemia
- Smokers over 35 years old
- A history of or active breast cancer

- Diabetes
- Heart disease
- Liver disease

BENEFITS

- **Trust Levo** can protect from pregnancy, when taken correctly and regularly
- The added iron can help overcome lethargy sometimes experienced during cyclical bleed
- Short acting contraception meaning conception can occur rapidly on stopping **Trust Levo**
- Simple and easy to use
- Regulates menstrual cycle

COMMON SIDE EFFECTS

Most side effects are mild and settle after a few weeks of regular use and these include:

- Spotting or bleeding at intervals
- Breast tenderness
- Nausea or vomiting
- Headache

Rx
Desogestrel & Ethinylestradiol
Tablets USP

Trust Deso[®]

3rd Generation
Birth Control Pills



Trust Deso is a 21 day cycle oral contraceptive pill containing (Desogestrel 0.15 mg and Ethinylestradiol 0.02 mg). In addition to pregnancy prevention, **Trust Deso** can reduce heavy menstrual bleeding (menorrhagia) and painful menstrual cramps (dysmenorrhea).



Pack Size 10 packs per box

Helps in Painful Menstrual Periods

*conditions applied, speak with a doctor

HOW IT WORKS

Trust Deso works by inhibiting the follicle-stimulating hormone which leads the mode of action also results in a thinning of the endometrial lining of the uterus thus making periods lighter, shorter and less painful.

HOW TO USE

1 tablet should be taken orally at the same time every day or at least within the same 12 hour window for 21 days beginning from day 1 of menstrual cycle (first bleed day of period) followed by a 7 day break. Commencing the next pack after the 7 day break or as otherwise directed by a healthcare provider.

HELPS WITH

- Pregnancy prevention
- Heavy menstruation / painful menstruation / premenstrual syndrome (PMS)

WHO CANNOT USE

Trust Deso should not be used by women with certain health conditions which include: liver disease, breast cancer, hypertension, venous thromboembolism or vascular heart disease.

BENEFITS

- **Trust Deso** can improve physical, emotional, social and psychological quality of life by offering a long-lasting solution to heavy and painful menstruation.
- Effective in preventing pregnancy
- Fertility returns (meaning you can get pregnant again) a few days after you stop taking **Trust Deso**
- Offers relief from premenstrual syndrome (PMS)
- Does not cause water retention or weight gain

COMMON SIDE EFFECTS

Most side effects are mild and settle after a few weeks of regular use, including:

- Spotting or bleeding at intervals
- Breasts may become tender and sore to touch
- Nausea or vomiting
- Headache

Rx Medroxyprogesterone Acetate
Injection IP

Trust Pari[®]
Contraceptive Injection



Trust Pari is a contraceptive injection containing a progestin (150 mg Medroxyprogesterone acetate). It is injected intramuscularly (usually the upper arm or buttocks) by a healthcare professional every 3 months. It is also known as DMPA-IM.



Packing Retail Box - 12 packs per box
Wholesale Box - 20 Vials per box

One Injection And Freedom For 3 Months

*conditions applied, speak with a doctor

HOW IT WORKS

Trust Pari prevents pregnancy by:

- Suppressing ovulation (release of an egg)
- Thickening the cervical mucus, so sperm cannot reach the egg
- Thinning the lining of the uterus to prevent implantation
- Provides 12 weeks of contraception from 1 single injection

WHO CAN USE

Trust Pari can be used by women who:

- Cannot use combined oral contraceptives
- Are breast feeding (infant over the age of 6 weeks old)
- Who have just completed an abortion or miscarriage

WHO CANNOT USE

Trust Pari is not recommended for women who have:

- Severe hypertension, ischemic heart disease or history of stroke
- A history of or active breast cancer, liver cancer or severe liver disease
- Vascular disease or acute deep venous thrombosis

BENEFITS

- Effective pregnancy prevention
- Discrete method of contraception
- Reliable contraception method for women who cannot take estrogen

COMMON SIDE EFFECTS

Some women may experience side effects following administration of **Trust Pari**. They usually subside after a few weeks and ease with subsequent dose injection every 12 weeks.

Side effects may include:

- Initially heavier period bleeding but can decrease to no period bleed
- Menstrual changes
- Weight changes
- Headache
- Mood changes
- Decreased desire to have sex (libido)

Trust Urvashi[®]
Intrauterine Contraceptive Device
Cu T 380A



Trust Urvashi[®]
Intrauterine Contraceptive Device
Cu375 Standard



Trust Urvashi is an Intrauterine Contraceptive Devices (IUD), a long term contraception device that can prevent pregnancy for 3 to 10 years. It is a reversible contraceptive method that is 99.9% effective. It is inserted into the uterus by a trained healthcare provider.



Pack Size 6 packs per box
50 packs per box

One Insertion And Freedom For 3-10 Years

*conditions applied, speak with a doctor

HOW IT WORKS

Trust Urvashi contains copper which provides a hostile environment for sperm thereby preventing it from fertilizing the egg.

WHO CAN USE

Trust Urvashi can be used by most women, including:

- Women who want a discrete and easy to use contraceptive
- Breast-feeding mothers
- Smokers or those with diabetes or hypertension
- Those who cannot or don't want to use hormonal contraceptives

WHO CANNOT USE

Trust Urvashi should not be used by women with certain health conditions including: a history of cervical cancer, endometrial disease, uterine prolapse, unknown vaginal bleeding or an untreated vaginal infection.

BENEFITS

- No hormone related side effects, e.g. headaches, mood swings, single depressions
- Starts to work effective immediately after insertion
- Can be removed at any time with ability to conceive returning quickly
- Can be used as an emergency contraception if inserted within 5 days of unprotected sex
- Prevents pregnancy for 3 to 10 years depending on the type of **Trust Urvashi** is inserted (Cu 375 & Cu T 380A)

COMMON SIDE EFFECTS

Some women may experience side effects following IUCD insertion. They usually go away after a few weeks and may include:

- Irregular and/or heavy periods
- Menstrual cramps
- Spotting between periods
- Abnormal vaginal discharge


MEET THE UNIQUE FEATURES ON THE URVASHI IUCD VARIANTS

Trust Urvashi Cu375 & Trust Urvashi Cu T 380A

- Long term contraception for 3-10 Years
- Suitable for nulliparous (women who have not previously been pregnant) and those with a history of pregnancy
- Hormone free and effective immediately,
- Even suitable as a method of emergency contraception
- Can be removed prematurely if pregnancy is desired with fertility returning immediately

Rx Misoprostol Tablets IP 200 mcg

Trust Miso®

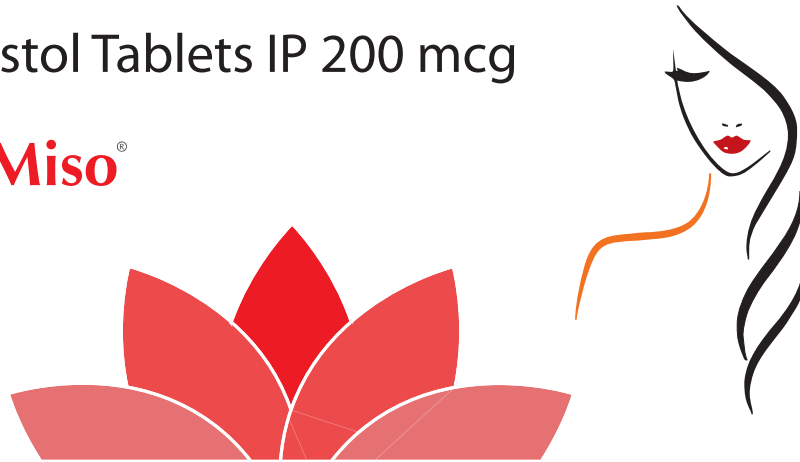

Trust Miso is tablets containing Misoprostol 200 mcg that is used in the management of obstetric and gynaecological conditions. These include prevention and treatment of postpartum haemorrhage (PPH), miscarriage/post-abortion care, cervical ripening and labor induction.



Pack Size 10 packs per box

Termination of Pregnancy up to 9 Weeks From Last Menstrual Cycle

*conditions applied, speak with a doctor

HOW IT WORKS

In the uterus: Increases the amplitude and frequency of contraction to expel the contents of the uterus.

In the cervix: Breaks down the cervical connective tissue allowing it to become soft, flexible and capable of dilation.

HOW TO USE

Indication	WHO Recommended Method of Administration
To prevent postpartum haemorrhage (PPH)	2 tablets (400 mcg) or 3 tablets (600 mcg) orally immediately after delivery of the baby (or in the case of multiple births, the last baby).
Treatment of incomplete abortion/ miscarriage < 13 weeks	3 tablets (600 mcg) orally or 2 (400 mcg) tablets sublingually.
Treatment of incomplete abortion/ miscarriage > 13 weeks	Repeat doses of 2 tablets (400 mcg) sublingually, vaginally or buccally every 3 hours.
Induced medical abortion <12 weeks	4 tablets (800 mcg) administered vaginally, sublingually or buccally.
Induced medical abortion >12 weeks*	Repeat doses of 2 tablets (400 mcg) administered vaginally, sublingually or buccally every 3 hours.

* up to the gestational age limit prescribed by law

Rx Combipack of Mifepristone & Misoprostol Tablets

**COMBI
CLEAR**

1x mifepristone
200 mg
4x misoprostol
200 mcg



This Pack Contains Treatment for one
Medical Abortion (early pregnancy)

COMBI CLEAR is a combination pack containing 1 tablet of Mifepristone 200 mg and 4 tablets of Misoprostol 200 mcg, a World Health Organization (WHO) recommended regimen for medical abortion and miscarriage management without the need for invasive surgical procedures.

COMBI CLEAR follows International Pharmacopeia Quality Standards



Pack Size 10 packs per box

Termination of Pregnancy up to 9 Weeks From last Menstrual Cycle

*conditions applied, speak with a doctor

HOW TO USE

COMBI CLEAR should be taken following consultation with and direction from a health care provider.

Note: These guidelines are included for information only. **COMBI CLEAR** should only be used as per the instructions of a registered medical practitioner.

Indication	WHO Recommended Method of Administration
Medical abortion < 12 weeks*	Single tablets (200 mg) mifepristone orally, followed 24-48 hours later by 4 tablets (800 mcg) misoprostol vaginally, sublingually or buccally.
Medical abortion > 12 weeks*	Single tablets (200 mg) mifepristone orally, followed 24-48 hours later by 2 tablets (400 mcg) misoprostol vaginally, sublingually or buccally. Repeat doses of misoprostol can be considered when needed to achieve success of the abortion process. Healthcare providers should use clinical judgement to decide the maximum number of doses of misoprostol required.

* up to the gestational age limit prescribed by law

Rx
Combipack of Mifepristone
& Misoprostol Tablets

safe-t kit



safe-t kit is a combination pack containing 1 tablet of Mifepristone 200 mg and 4 tablets of Misoprostol 200 mcg, a World Health Organization (WHO) recommended regimen for medical abortion and miscarriage management without the need for invasive surgical procedures.



Combipack of Mifepristone 200 mg and Misoprostol 200 mcg tablets

Pack Size 20 packs per box

Termination of Pregnancy up to 9 Weeks From Last Menstrual Cycle

*conditions applied, speak with a doctor

HOW TO USE

Safe-t kit should be taken following a consultation and direction from a health care provider.

Note: These guidelines are included for information only. **Safe-t kit** should only be used as per the instructions of a registered medical practitioner.


Indication	WHO Recommended Method of Administration
Medical abortion < 12 weeks*	1 tablets (200 mg) mifepristone orally, followed 24-48 hours later by 4 tablets (800 mcg) misoprostol vaginally, sublingually or buccally.
Medical abortion > 12 weeks*	1 tablets (200 mg) mifepristone orally, followed 24-48 hours later by 2 tablets (400 mcg) misoprostol vaginally, sublingually or buccally. Repeat doses of misoprostol can be considered when needed to achieve success of the abortion process. Healthcare providers should use clinical judgement to decide the maximum number of doses of misoprostol required.

* up to the gestational age limit prescribed by law

Ipas MVA Plus[®]



Ipas MVA Plus is a manual vacuum aspiration kit used for evacuating uterine contents providing a safe, effective and affordable method for termination of pregnancy for uterine size up to 12 weeks of gestation. It is a preferred method for uterine evacuation recommended by World Health Organization (WHO) and International Federation of Gynecology and Obstetrics (FIGO).

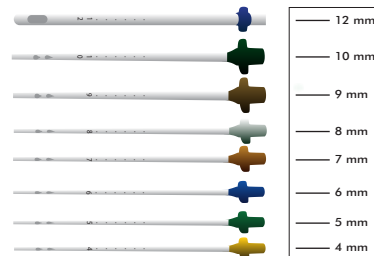

Selecting the appropriate cannulae size (accessories to MVA device)

Gestational age	Suggested cannulae size
4 to 6 weeks	4 – 7 mm
7 to 9 weeks	5 – 10 mm
9 to 12 weeks	8 – 12 mm

CLINICAL INDICATIONS

- Treatment of incomplete abortion for uterine sizes up to 12 weeks since date of last menstrual period.

DISCREET AND PATIENT FRIENDLY

- Enables faster recovery time compared to surgical procedures

SAFE AND EFFECTIVE

- 98% effective with low complication rates*
- Safe alternative to sharp curettage

AFFORDABLE AND VERSATILE

- Reduces procedural costs by performing procedures in an outpatient setting
- Double valve buttons for greater pressure
- It is portable and ideal for environments with no electricity or with space constraints
- Low initial investment and per-use cost

WHEN TO REPLACE

- After 25 uses
- The cylinder or cannula is brittle
- Presence of tissue deposits
- The aspirator or cannula is broken
- The aspirator no longer holds vacuum

* In less than 2% of cases one or more of the following can occur during or after the procedure: uterine or cervical injury, pelvic infection, vaginal reaction, incomplete evacuation or acute hematometra

Pack Size 1 Ipas MVA Plus Kit = 1 Aspirator + 8 Easy Grip colour coded cannulae: size (4, 5, 6, 7, 8, 9, 10 & 12 mm)